



1515 Dekalb Pike
Suite 100A
Blue Bell, PA 19422
610-277-1990 x 151

**THE EZ BACK IS
DESIGNED WITH
COMFORT IN MIND.**

IT ALLOWS PROTECTED
MOTION WITH ITS
MOLDABLE AND
ADJUSTABLE POSTERIOR
PANELS TO AID IN
HEALING WHILE LIVING
LIFE WITH CONTROLLED
MOTION, DECREASED
PAIN AND WITH
OPTIMAL COMFORT.



DITH™

The EZ BACK from Doctor in the House

The EZ BACK from Doctor in the House is a unique LSO brace developed by an orthopedic surgeon to help patients heal from back injuries, fractures and sprains without surgery and to help rehabilitate post operative back patients to optimal health and lifestyle.

docinthehouse.com | 610-277-1990 x 151

Designed With Comfort In Mind

DITH EZ BACK Treats:

- Fractures
- Ligament Injuries
- Disc Herniations & DJD of Spine
- Arthritis & Degenerative Arthritis
- Osteoarthritis and Rheumatoid Arthritis
- Osteoporosis
- Sacral and Sacroiliac Fractures and Ligament Injuries
- Lumbar Inter-segmental Instability
- SI Joint Injury
- Compression Fractures
- Postop Lumbar Spine Surgery



Our braces and splints are designed with comfort in mind. They allow maximum support and dynamic healing through protected motion, while decreasing your pain as you live your life.

Key to the EZ Back are its optional, adjustable, moldable & removable posterior panels that literally allow your brace to fit you like a glove while providing soothing comfort and posterior thoracic, lumbar and sacral support.

Your body knows how to heal and our team at Doctor in the House wants to help you do just that!



Patient Friendly Design

- Uniform compression & lightweight
- Adjustable, moldable, optional & removable posterior panels provide maximum comfortable support to the sacrum and lumbar vertebrae allowing ease of living life
- Easy size adjustment for optimal comfort
- Easily adjustable to conform to all body types
- Patient friendly especially for arthritic patients
- Ability to wear under clothes
- Can be adjusted and molded by your doctor or therapist

How To Apply

1. Open the belt and adjust the lumbosacral support panels to your area of maximum desired support.
2. Insert fingers in loop on panels and snug to fit, then simply press to fasten the velcro in the front.
3. Adjust to comfort.



BELT ALONE



BELT WITH PANELS



You're all set!

Optional belt extender panel



The Best Back Brace Around You!

Clinical Indications

The unique support provided by the EZ BAcK is attained through the adjustable & removable posterior panels. This LSO brace is specifically designed to support the lumbosacral spine and allows comfortable support but also disc space unloading to decrease disc and nerve root compression. This provides comfort as well for muscle spasm, fractures and arthritis.

This lumbosacral orthotic device provides comfortable support, allowing protected motion, enhancing muscle conditioning while protecting against paraspinal spasm experienced due to nerve root compression, compression fractures, osteoporosis, disc degeneration, as well as herniation, spinal stenosis, and sacroiliac or sacrococcygeal ligament injury.

Important Information to Read

As with any medical device if you develop any new pain symptoms, you should contact your treating medical professional immediately. Symptoms such as pain, numbness, tingling, burning, chafing or irritation of the skin and progressive stiffness are not normal.

Doctor in the House™ makes every effort to ensure the accuracy and completeness of information provided with this product. Doctor in the House™ disclaims any liability or responsibility for injury or damage to persons or property which is incurred as a consequence, directly or indirectly, of the use and application of The DITH™ EZ Back. It is the users responsibility to know and follow local care protocols provided by the medical advisors directing the system to which he or she belongs. Medicine is an ever changing science. In view of the possibility of human error or changes in medical science, neither the founders, employees or any other party who has been involved in the preparation or production of The DITH™ Miracle Back Pro product warrant that the design and information contained herein is in every respect accurate or complete and they are not responsible for any errors, omissions, defects or injuries from the results obtained from the use of information or DITH™ EZ Back.

Warranty Information

Users are encouraged to confirm the information contained herein with other sources. Your DITH™ Miracle Back Pro is warranted against defects found in material and/or workmanship for six months when used in accordance with generally accepted medical use and in accordance with the instructions provided by DITH.

Alteration to, misuse of, or failure to maintain use pursuant with the provided directions from DITH and your medical treating professional shall void this warranty. Normal wear and tear during the usage of a product is not considered a defect.

If it is determined that a return or replacement of the product is appropriate, instructions will be provided as to how to do so. If the product is unused and a request is made for return, it must be in the original packaging and in new resalable condition or no refund can be given. No merchandise is returnable without first obtaining company authorization.

www.docinthehouse.com

Patent Pending

1515 Dekalb Pike
Suite 100A
Blue Bell, PA 19422
610-277-1990 x 151