

DITH™




**THE BACK BANDIT  
ULTIMA IS DESIGNED  
WITH COMFORT IN MIND.**

It allows rigid 4 panel-triplanar support, offering protected motion for the entire back with its customizable, heat moldable panels and easy adjustment to aid in healing while living life with protected motion, **decreased pain** and with **optimal comfort**.

## The Back Bandit Ultima from Doctor in the House

The Back Bandit Ultima from Doctor in the House is a unique TLSO brace developed by an orthopedic surgeon to help patients heal from back injuries, fractures and sprains without surgery and to help rehabilitate post operative back patients to optimal health and lifestyle.

## Designed With Comfort In Mind

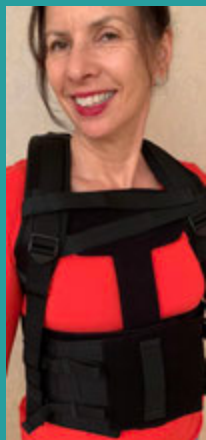

### DITH Back Bandit Ultima Treats:

- Fractures
- Ligament Injuries
- Disc Herniations & DJD of Spine
- Arthritis & Degenerative Arthritis
- Upper Back & Rib Fractures
- Osteoporosis
- Thoracic Kyphosis and Scoliosis
- Lumbar Inter-segmental Instability
- Lumbar Fractures
- Postop Lumbar and Thoracic Spine Surgery



Our braces and splints are designed with comfort in mind. They allow maximum support and dynamic healing through protected motion, while decreasing your pain as you live your life.

Key to the Back Bandit Ultima are its 4 rigid panels, posterior, lateral and anterior providing optimal support to the Lumbosacral, Lumbar and Thoracic Spine. These panels can be customized, adjusted, molded and cut to fit you like a glove while providing soothing comfort and full spine support while healing.

Your body knows how to heal and our team at Doctor in the House wants to help you do just that!



PATENT PENDING

## Patient Friendly Design

- Uniform triplanar support and lightweight
- 4 rigid supporting panels allow anterior, lateral and posterior full support from SI joint to Scapular spine
- Moldable and Customizable for optimal comfort and support
- Patient friendly design by an orthopaedic surgeon for his patients

## How To Apply


Once your Back Bandit Ultima has been fitted and customized for you, It's as easy as 1,2,3

1. Open the bottom belt and slip your arms through the straps like the sleeves of a shirt.
2. Apply the Anterior Chest Panel straps to the upper thoracic straps and secure to the belt on the inside anterior panel.
3. Insert fingers in loop on panels and snug to fit, then simply press to fasten the velcro in the front and adjust the thoracic straps to comfort.

1




2

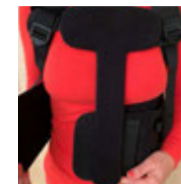

3



You're all set!

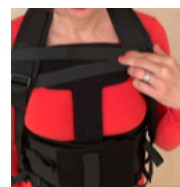

Optional belt extender panel

# The Best Back Brace Around You!

## *Clinical Indications*

The unique support provided by the Back Bandit Ultima allows customizable support offered through the 4 rigid panels. The design not only supports the entire thoracic and lumbosacral spine, it also provides unique support to the paraspinous musculature further limiting and balancing rotational deformity of the spine and thorax along with the rib cage, providing even greater comfort for the pain and muscle spasm associated with fractures and discogenic disease.

This device provides anterior/posterior TLSO support plus it restricts and controls motion in the sagittal, coronal and transverse planes, protecting against paraspinous spasm experienced due to nerve root compression, lumbar inter-segmental instability, spondylosis, spondylitis and spondylolisthesis, post-operative lumbar and thoracic spine surgery, thoracic and lumbar disc herniation, spinal nerve root compression thoracic or lumbar disc desiccation, spinal stenosis, rib fractures, ligament injury & fracture-dislocations, costochondritis, thoracolumbar musculoligamentous injury, spinal fusion surgery, osteoarthritis and rheumatoid arthritis, sacroiliitis, intractable back pain, metastatic spine lesions and spinal tumor, stabilization, osteomyelitis of the spine, transverse process & vertebral body fractures, thoracic and lumbar strain and sprain, sacrum and coccygeal fractures and strains, sacrococcygeal ligament injury and fractures/dislocations, iliopsoas strain lumbago and lumbar back pain with lumbosacral instability, osteoporosis, thoracic kyphosis and scoliosis, lumbar and thoracic spine compression fractures

## *Important Information to Read*

As with any medical device if you develop any new pain symptoms, you should contact your treating medical professional immediately. Symptoms such as pain, numbness, tingling, burning, chafing or irritation of the skin and progressive stiffness are not normal.

Doctor in the House™ makes every effort to ensure the accuracy and completeness of information provided with this product. Doctor in the House™ disclaims any liability or responsibility for injury or damage to persons or property which is incurred as a consequence, directly or indirectly, of the use and application of The DITH™ Back Bandit Ultima. It is the user's responsibility to know and follow local care protocols provided by the medical advisors directing the system to which he or she belongs. Also, it is the user's responsibility to stay informed of changes in the treatment of upper extremity injuries. Medicine is an ever changing science. In view of the possibility of human error or changes in medical science, neither the founders, employees or any other party who has been involved in the preparation or production of The DITH™ Back Bandit Ultima product warrant that the design and information contained herein is in every respect accurate or complete and they are not responsible for any errors, omissions, defects or injuries from the results obtained from the use of information or DITH™ Back Bandit Ultima.

## *Warranty Information*

Users are encouraged to confirm the information contained herein with other sources. Your DITH™ Back Bandit Ultima is warranted against defects found in material and/or workmanship for six months when used in accordance with generally accepted medical use and in accordance with the instructions provided by DITH.

Alteration to, misuse of, or failure to maintain use pursuant with the provided directions from DITH and your medical treating professional shall void this warranty. Normal wear and tear during the usage of a product is not considered a defect.

If it is determined that a return or replacement of the product is appropriate, instructions will be provided as to how to do so. If the product is unused and a request is made for return, it must be in the original packaging and in new resalable condition or no refund can be given. No merchandise is returnable without first obtaining company authorization.

[www.docinthehouse.com](http://www.docinthehouse.com)  
Patents Pending

1515 Dekalb Pike  
Suite 100A  
Blue Bell, PA 19422  
610-277-1990 x 151