



1515 Dekalb Pike
Suite 100A
Blue Bell, PA 19422
610-277-1990 x 151

**THE MIRACLE BACK
ULTRA IS DESIGNED
WITH YOUR
COMFORT IN MIND**

**ANTERIOR, LATERAL &
POSTERIOR PANELS
AID IN HEALING AND
CONTROL PAIN WHILE
LIVING LIFE WITH
PROTECTED MOTION
IN OPTIMAL COMFORT**



DITH™

The Miracle Back Ultra from Doctor in the House

The Miracle Back Ultra from Doctor in the House is a unique LSO brace developed by an orthopedic surgeon to help patients heal from back injuries, fractures and sprains without surgery and to help rehabilitate post operative back patients to optimal health and lifestyle.

docinthehouse.com | 610-277-1990 x 151

HCPCS L0637

Designed With Comfort In Mind

The Miracle Back Ultra Treats:


- Fractures
- Ligament Injuries
- Disc Herniations & DJD of Spine
- Arthritis & Degenerative Arthritis
- Osteoarthritis and Rheumatoid Arthritis
- Osteoporosis
- Sacral and Sacroiliac Fractures and Ligament Injuries
- Lumbar Inter-segmental Instability
- Spondylosis, Spondylitis and
- Compression Fractures
- Postop Lumbar Spine Surgery



Our braces and splints are designed with your comfort in mind. They allow maximum support and dynamic healing through protected motion, while decreasing your pain as you live your life.

Key to the Miracle Back Ultra are its optional, adjustable & removable posterior, anterior and lateral orthopaedic frame/panels that literally allow your brace to fit you like a glove while providing soothing comfort and posterior thoracic, lumbar and sacral support.

Your body knows how to heal and our team at Doctor in the House wants to help you do just that!

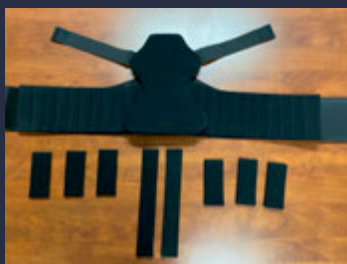

PATENT PENDING

Patient Friendly Design

- Uniform compression & lightweight
- Adjustable, optional & removable posterior, anterior and lateral panels provide maximum comfortable support from the sacrum to T-9 vertebrae allowing ease of living life
- Increased intracavitary pressure unloads disc compression forces
- Easy size adjustment for optimal comfort
- Easily adjustable to conform to all body types
- Patient friendly especially for arthritic patients
- Ability to wear under clothes
- Can be adjusted and molded by your doctor or therapist

How To Apply

1. Open the belt and adjust the large posterior frame/panel as well as side panels to your area of maximum desired support.
2. Insert fingers in loop on panels and snug to fit, then simply press to fasten the velcro in the front.
3. Adjust the upper back panel snugging straps to comfort.




Snugging straps for large back panel



You're all set!

Optional belt extender panel

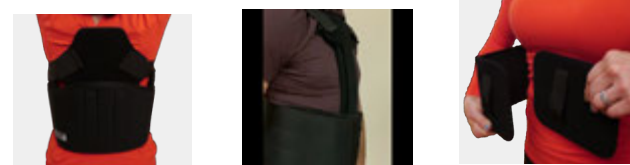

The Best Back Brace Around You!

Clinical Indications

The unique support provided by the Miracle Back Ultra is attained through the versatility offered by the combination of the adjustable & removable posterior panel and the lateral and anterior frame/panels. This lumbosacral orthotic (LSO) brace is specifically designed to support the lumbosacral spine and allows comfortable support but also disc space unloading to decrease disc and nerve root compression. This pressure offloading support provides comfort as well for muscle spasm, fractures and arthritis.

This LSO device provides comfortable anterior, posterior and lateral support especially in the sagittal and coronal planes of motion while protecting against paraspinal spasm experienced due to nerve root compression, compression fractures, osteoporosis, disc degeneration, as well as herniation, spinal stenosis, and sacroiliac or sacrococcygeal ligament injury.

Important Information to Read

As with any medical device if you develop any new pain symptoms, you should contact your treating medical professional immediately. Symptoms such as pain, numbness, tingling, burning, chafing or irritation of the skin and progressive stiffness are not normal.

Doctor in the House™ makes every effort to ensure the accuracy and completeness of information provided with this product. Doctor in the House™ disclaims any liability or responsibility for injury or damage to persons or property which is incurred as a consequence, directly or indirectly, of the use and application of The Miracle Back Ultra. It is the user's responsibility to know and follow local care protocols provided by the medical advisors directing the system to which he or she belongs. Medicine is an ever-changing science. In view of the possibility of human error or changes in medical science, neither the founders, employees or any other party who has been involved in the preparation or production of the Doctor in the House™ Miracle Back Ultra product warrant that the design and information contained herein is in every respect accurate or complete and they are not responsible for any errors, omissions, defects or injuries from the results obtained from the use of information or your Miracle Back Ultra.

Warranty Information

Users are encouraged to confirm the information contained herein with other sources. Your Miracle Back Ultra is warranted against defects found in material and/or workmanship for six months when used in accordance with generally accepted medical use and in accordance with the instructions provided by Doctor in the House™.

Alteration to, misuse of, or failure to maintain use pursuant with the provided directions from Doctor in the House™ and your medical treating professional shall void this warranty. Normal wear and tear during the usage of a product is not considered a defect.

If it is determined that a return or replacement of the product is appropriate, instructions will be provided as to how to do so. If the product is unused and a request is made for return, it must be in the original packaging and in new resalable condition or no refund can be given. No merchandise is returnable without first obtaining company authorization.

www.docinthehouse.com Patent
Patents Pending

1515 Dekalb Pike
Suite 100A
Blue Bell, PA 19422
610-277-1990 x 151