



1515 Dekalb Pike  
Suite 100A  
Blue Bell, PA 19422  
610-277-1990 x 151

THE BACK BANDIT PRO IS DESIGNED WITH COMFORT IN MIND, IT ALLOWS PROTECTED MOTION FOR THE ENTIRE BACK WITH ITS ADJUSTABLE POSTERIOR ORTHOPAEDIC PANEL TO AID IN HEALING WHILE LIVING LIFE WITH PROTECTED MOTION, DECREASED PAIN AND WITH OPTIMAL COMFORT.



DITH™

## The Back Bandit Pro from Doctor in the House

The Back Bandit Pro from Doctor in the House is a unique TLSO brace developed by an orthopedic surgeon to help patients heal from back injuries, fractures and sprains without surgery and to help rehabilitate post operative back patients to optimal health and lifestyle.

[docinthehouse.com](http://docinthehouse.com) | 610-277-1990 x 151

HCPCS L0457

## Designed With Comfort In Mind



### DITH Back Bandit Pro Treats:

- Fractures
- Ligament Injuries
- Disc Herniations & DJD of Spine
- Arthritis & Degenerative Arthritis
- Upper Back & Rib Fractures
- Osteoporosis
- Thoracic Kyphosis and Scoliosis
- Lumbar Inter-segmental Instability
- Lumbar Fractures
- Postop Lumbar and Thoracic Spine Surgery



Our braces and splints are designed with comfort in mind. They allow maximum support and dynamic healing through protected motion, while decreasing your pain as you live your life.

Key to the Back Bandit Pro are its optional adjustable and removable posterior orthopaedic panel and thoracic straps that literally allow your brace to fit you like a glove while providing soothing comfort and posterior thoracic, lumbar and sacral support!

Your body knows how to heal and our team at Doctor in the House wants to help you do just that!



PATENT PENDING

## Patient Friendly Design

- Uniform compression & lightweight
- The posterior panel to give maximum spinal/paraspinal support from scapular spine to sacrum, as well as the paraspinal muscles allowing easy healing
- Posterior panel and thoracic straps provide 3 point posterior, lateral and anterior support
- Easy size adjustment for optimal comfort at the thoracic and lumbar segments
- Patient friendly
- Ability to wear under clothes

## How To Apply

1. Open the bottom belt and slip your arms through the straps like the sleeves of a shirt.
2. Insert fingers in loop on panels and snug to fit, then simply press to fasten the velcro in the front
3. Adjust the thoracic straps to comfort.

1




2



3



You're all set!

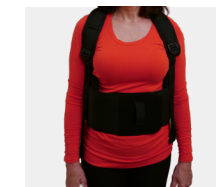

Optional belt extender panel

# The Best Back Brace Around You!

## *Clinical Indications*

The unique support provided by the Back Bandit Pro allows customizable support offered through the adjustable & removable posterior panel and thoracic straps. The design not only supports the thoracic and lumbosacral spine, it also provides unique support to the paraspinal musculature further limiting and balancing rotational deformity of the spine and thorax along with the rib cage, providing even greater comfort for the pain and muscle spasm associated with fractures and discogenic disease.

This device provides anterior/posterior TLSO support plus it restricts and controls rotation and side bending deformity in the sagittal plane, protecting against paraspinal spasm experienced due to nerve root compression, compression fractures, osteoporosis, disc desiccation as well as herniation, spinal stenosis, rib fractures, sternochondral/costochondral fracture-dislocations, scoliosis, excessive kyphosis and thoracolumbar musculoligamentous injury.

## *Important Information to Read*

As with any medical device if you develop any new pain symptoms, you should contact your treating medical professional immediately. Symptoms such as pain, numbness, tingling, burning, chafing or irritation of the skin and progressive stiffness are not normal.

Doctor in the House™ makes every effort to ensure the accuracy and completeness of information provided with this product. Doctor in the House™ disclaims any liability or responsibility for injury or damage to persons or property which is incurred as a consequence, directly or indirectly, of the use and application of The DITH™ Back Bandit Pro. It is the users responsibility to know and follow local care protocols provided by the medical advisors directing the system to which he or she belongs. Also, it is the reader's responsibility to stay informed of changes in the treatment of upper extremity injuries. Medicine is an ever changing science. In view of the possibility of human error or changes in medical science, neither the founders, employees or any other party who has been involved in the preparation or production of The DITH™ Back Bandit Pro product warrant that the design and information contained herein is in every respect accurate or complete and they are not responsible for any errors, omissions, defects or injuries from the results obtained from the use of information or DITH™ Back Bandit Pro.

## *Warranty Information*

Users are encouraged to confirm the information contained herein with other sources. Your DITH™ Back Bandit Pro is warranted against defects found in material and/or workmanship for six months when used in accordance with generally accepted medical use and in accordance with the instructions provided by DITH.

Alteration to, misuse of, or failure to maintain use pursuant with the provided directions from DITH and your medical treating professional shall void this warranty. Normal wear and tear during the usage of a product is not considered a defect.

If it is determined that a return or replacement of the product is appropriate, instructions will be provided as to how to do so. If the product is unused and a request is made for return, it must be in the original packaging and in new resalable condition or no refund can be given. No merchandise is returnable without first obtaining company authorization.

[www.docinthehouse.com](http://www.docinthehouse.com)  
Patent Pending HCPCS L0457

**1515 Dekalb Pike  
Suite 100A  
Blue Bell, PA 19422  
610-277-1990 x 151**