

APPLICATION INSTRUCTIONS



ROGUE WRAP

Size: UNIVERSAL

Item # KB0000-126

Sugg. HCPCS Code: L1832 ad L1833

- Universally Sized - Fits a wide variety of patients.
- Unique wrap around closures for easy fitting.
- Hidden hinge provides added support.
- Lightweight design.

-: HOW TO APPLY :-

1. Unfasten the closure straps. Place the brace around the knee so that hinges line up with the center of the kneecap on both sides of the leg, and secure the top closure.
2. Secure the bottom closure using the velcro tab. Keep the hinges aligned with the center of the knee and maintain a tight, snug fit.

-: CARE INSTRUCTIONS: :-

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water. Air dry away from heat.

DO NOT TWIST OR WRING - DO NOT TUMBLE DRY - DO NOT BLEACH

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation or swelling.