



APPLICATION INSTRUCTIONS



THE MIRACLE KNEE

Size: UNIVERSAL

Sugg. HCPCS Code: L1832 & L1833

- Universal Size - Fits a wide variety of patients.
- Unique wrap around closures for easy fitting.
- Dual hinged support.
- Lightweight design open front and back.
- Dual wrap design for easy customization

-: HOW TO APPLY :-

1. Unfasten the front closure straps. Place the brace around the knee so that hinges line up with the center of the kneecap on both sides of the leg, and secure the top closure.
2. Secure the bottom closure using the velcro tab. Keep the hinges aligned with the center of the knee and maintain a tight, snug fit.
3. Adjust posterior closures if needed for further customized fit and comfort.
4. Wrap on and apply suspension sleeve for extra support and to avoid brace slipping.
(see instructions included with the sleeve)

-: CARE INSTRUCTIONS: :-

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water. Air dry away from heat.

Do not bend or twist - air or delicate dry in dryer

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation or swelling.