



The Doctors Cervical Collar

Features:

1. Simple design, easy to operate.
2. Restricts bending, stretch and turn of the cervical vertebra effectively.
3. Special front and back adjustment knob for easy adjustment and sizing.
4. Adjustable sliding front posts and pillar for secure stability at just the right height
5. Convenient design allows inspection and treatment of the neck while wearing the collar.

How to use your Doctors Cervical Collar:



1. Turn Knobs on back of collar to the left to open for sizing
2. Detach the buckles top and bottom on the side of the collar
3. Test the collar for proper height and temporarily adjust the size around the neck
4. Secure the side buckles and check to be sure the collar is not too tight around the neck



5. Turn the 2 knobs on the front slowly to the left to adjust the inverted U-shape jaw posts to suitable height, fit the patient's jaw, then screw the rotary knobs to secure tightly.
6. Readjust the rear sizing knobs and be sure that the fit is comfortable and snug enough for support but not too loose that it may cause chafing.

You're all set!

This device is helpful in the treatment of arthritis, disc bulge or herniation, stable cervical spine fractures, post operative patients, rheumatoid arthritis, cervical sprain and strain and cervical radiculopathy.

Be sure you are under the care of a medical professional when using this device and use it per their instructions

1. Please use under the guidance of your doctor or medical professional
2. Check skin and or wounds regularly
3. Make sure to adjust the rotary knob slowly
4. Soft material can be washed with damp cloth and soft detergent, avoid acidic corrosive substances.
5. Avoid prolonged exposure to extreme heat as the orthoplastic may be deformed

Store in a cool place.