



721 Skippack Pike  
Building B, Suite 103  
Blue Bell, PA 19422  
610-277-1990 x 151

**THE MIRACLE BACK PRO II IS DESIGNED WITH COMFORT IN MIND. IT ALLOWS PROTECTED MOTION WITH ITS ADJUSTABLE POSTERIOR, LATERAL AND ANTERIOR FRAME/PANELS TO AID IN HEALING WHILE LIVING LIFE WITH CONTROLLED MOTION, DECREASED PAIN AND WITH OPTIMAL COMFORT.**



**DITH™**

## **The Miracle Back Pro II (HCPCS L0651) from Doctor in the House**

The Miracle Back Pro II from Doctor in the House is a unique LSO brace developed by an orthopedic surgeon to help patients heal from back injuries, fractures and sprains without surgery and to help rehabilitate post operative back patients to optimal health and lifestyle.

[docinthehouse.com](http://docinthehouse.com) | 610-277-1990 x 151

## Designed With Comfort In Mind



### DITH Miracle Back Pro II Treats:

- Fractures
- Ligament Injuries
- Disc Herniations & DJD of Spine
- Arthritis & Degenerative Arthritis
- Osteoarthritis and Rheumatoid Arthritis
- Osteoporosis
- Sacral and Sacroiliac Fractures and Ligament Injuries
- Lumbar Inter-segmental Instability
- Spondylosis, Spondylitis and
- Compression Fractures
- Postop Lumbar Spine Surgery



Our braces and splints are designed with comfort in mind. They allow maximum support and dynamic healing through protected motion, while decreasing your pain as you live your life.

Key to the Miracle Back Pro II and its optional, adjustable & removable posterior, anterior and lateral orthopaedic frame/panels that literally allow your brace to fit you like a glove while providing soothing comfort and posterior thoracic, lumbar and sacral support.

Your body knows how to heal and our team at Doctor in the House wants to help you do just that!




## Patient Friendly Design

- Uniform compression & lightweight
- Adjustable, optional & removable posterior, anterior and lateral panels provide maximum comfortable support from the sacrum to T-9 vertebrae allowing ease of living life
- Increased intracavitary pressure unloads disc compression forces
- Easy size adjustment for optimal comfort
- Easily adjustable to conform to all body types
- Patient friendly especially for arthritic patients
- Ability to wear under clothes
- Can be adjusted and molded by your doctor or therapist

## How To Apply


1. Open the belt and adjust the large posterior frame/panel as well as side panels to your area of maximum desired support.
2. Insert fingers in loop on panels and snug to fit, then simply press to fasten the velcro in the front.
3. Adjust the upper back panel snugging straps to comfort.

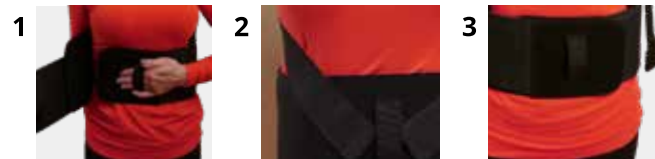


Snugging straps for large back panel



Optional reinforcing chest panel

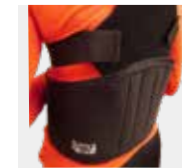

Optional belt extender panel

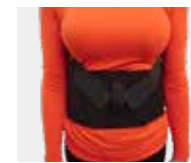

# The Best Back Brace Around You!

## *Clinical Indications*

The unique support provided by the Miracle Back II is attained through the adjustable & removable posterior and additionally from the lateral and anterior frame/panels. This LSO brace is specifically designed to support the lumbosacral spine and allows comfortable support but also disc space unloading to decrease disc and nerve root compression. This provides comfort as well for muscle spasm, fractures and arthritis.

This lumbosacral orthotic device provides comfortable anterior, posterior and lateral support especially in the sagittal and coronal planes of motion while protecting against paraspinous spasm experienced due to nerve root compression, compression fractures, osteoporosis, disc degeneration, as well as herniation, spinal stenosis, and sacroiliac or sacrococcygeal ligament injury.

## *Important Information to Read*

As with any medical device if you develop any new pain symptoms, you should contact your treating medical professional immediately. Symptoms such as pain, numbness, tingling, burning, chafing or irritation of the skin and progressive stiffness are not normal.

Doctor in the House™ makes every effort to ensure the accuracy and completeness of information provided with this product. Doctor in the House™ disclaims any liability or responsibility for injury or damage to persons or property which is incurred as a consequence, directly or indirectly, of the use and application of The DITH™ Miracle Back Pro II. It is the users responsibility to know and follow local care protocols provided by the medical advisors directing the system to which he or she belongs. Medicine is an ever changing science. In view of the possibility of human error or changes in medical science, neither the founders, employees or any other party who has been involved in the preparation or production of The DITH™ Miracle Back Pro II product warrant that the design and information contained herein is in every respect accurate or complete and they are not responsible for any errors, omissions, defects or injuries from the results obtained from the use of information or DITH™ Miracle Back Pro II.

## *Warranty Information*

Users are encouraged to confirm the information contained herein with other sources. Your DITH™ Miracle Back Pro II is warranted against defects found in material and/or workmanship for six months when used in accordance with generally accepted medical use and in accordance with the instructions provided by DITH.

Alteration to, misuse of, or failure to maintain use pursuant with the provided directions from DITH and your medical treating professional shall void this warranty. Normal wear and tear during the usage of a product is not considered a defect.

If it is determined that a return or replacement of the product is appropriate, instructions will be provided as to how to do so. If the product is unused and a request is made for return, it must be in the original packaging and in new resalable condition or no refund can be given. No merchandise is returnable without first obtaining company authorization.

www.docinthehouse.com  
Patent Pending

721 Skippack Pike  
Building B, Suite 103  
Blue Bell, PA 19422  
610-277-1990 x 151